Handstückablage VistaCam iX

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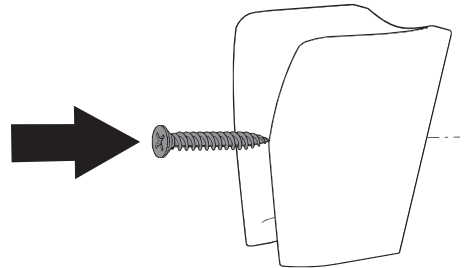
VistaCam iX Handpiece Holder

The handpiece holder can be stuck or screwed in position.

- Mount the handpiece holder near to where the handpiece will be used.
The length of the USB cable is 2.5 m.

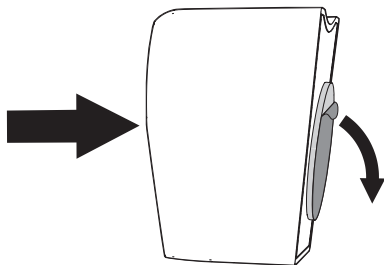
Do not place or store any magnetic strip cards or similar items in the vicinity of the handpiece holder.

The handpiece holder is fitted with a magnet which serves to switch the camera off when activated by the magnet sensor when the camera is replaced.

1 Sticking the handpiece holder in place

The surface to which the handpiece holder is to be stuck must be smooth, clean and free of any oil or grease.

- Peel off the protective foil.
- Press the handpiece holder firmly into place at the required position.

2 Screwing the handpiece holder in place

Use suitable materials according to the type of surface to which the handpiece holder is to be fixed. The delivery contents include an M3 x 20 oval headed screw and a 3.0 x 30 MDF screw.

- When using the oval headed screw: mark the position of the hole to be drilled and drill the required hole (\varnothing 3.2 mm).
- Position the handpiece holder and secure with the screw.