Vector Paro / Vector Scaler Instrumentenverrundung

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
Vector Paro / Vector Scaler Instrument filing

The following informations are additional to the Installation and Operating instructions.

Always read the Installation and Operating Instructions relevant to the appliance in addition, these contain important information describing Safety Instructions, Set-up, Electrical Connection, Disinfection, Cleaning etc.

Delivery Contents

Whetstone 9000-442-02E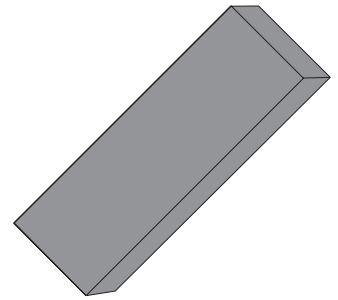


1 Instrument wear

Continued use of the instruments can lead to signs of wear:

- wear on one side only
- sharp edges on the sides or tiip of the instrument
- general signs of wear

2 Filing of instruments


- Place handpiece in position.
- Place the worn instrument into handpiece.
- The Fluid key of the base station must be switched on.
- Check that the fluid container is full.
- Place collector vessel ready.
- Switch on appliance using the Flex-foot switch.
- Hold the instrument at a slightly inclined angle during normal operation (2-3 LEDs lit) and apply light pressure against the whetstone.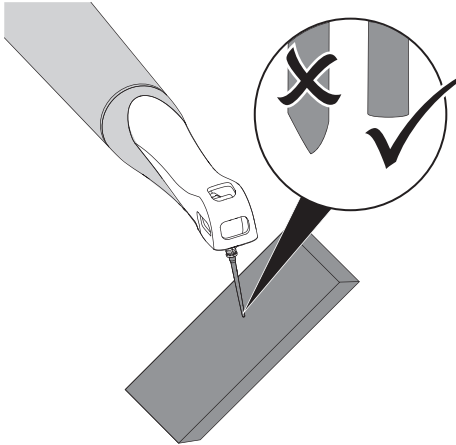

- To round the instrument surfaces rotate either the handpiece or the whetstone.
- Check that the instrument surfaces are sufficiently rounded. No sharp edges must be visible or felt.

3 Check instrument wear

Instrument wear can be checked using the Vector Toolcard:

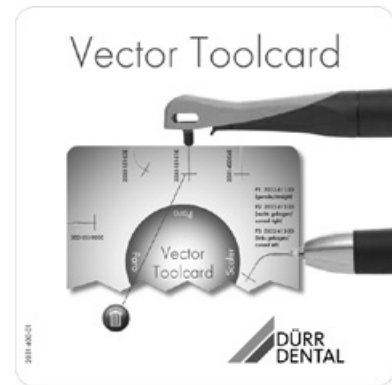- Place the handpiece against the Toolcard. If the instrument tip extends beyond the red marking then the instrument can still be used.
- If the instrument tip just reaches the red marking then the instrument shows signs of wear but can still be used.
- If the instrument tip does not reach the red marking then the instrument shows signs of increased wear and must be replaced.

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